H-MAX

Femoral Stem
Stelo femorale

**SURGICAL TECHNIQUE
TECNICA OPERATORIA**

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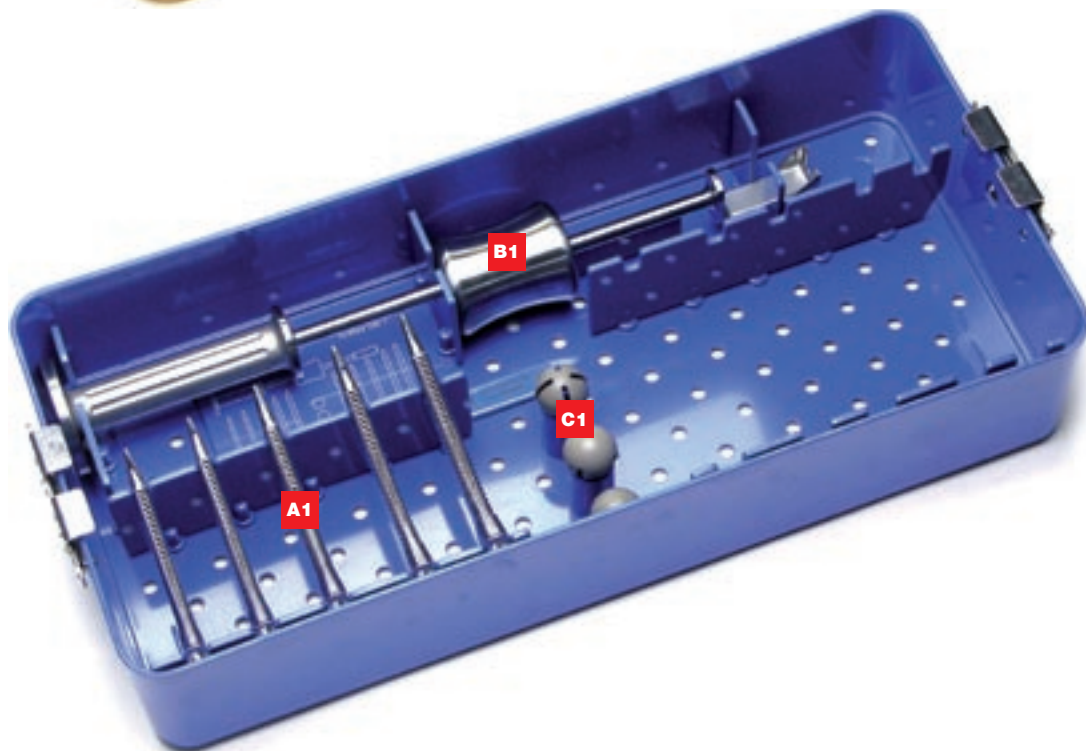
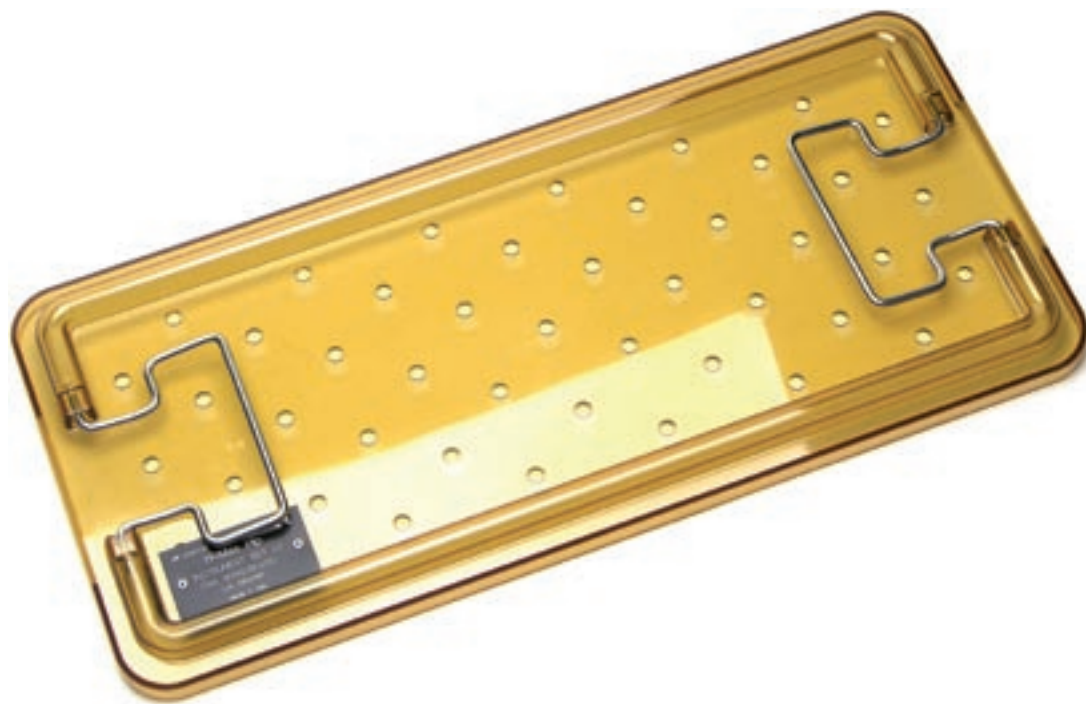
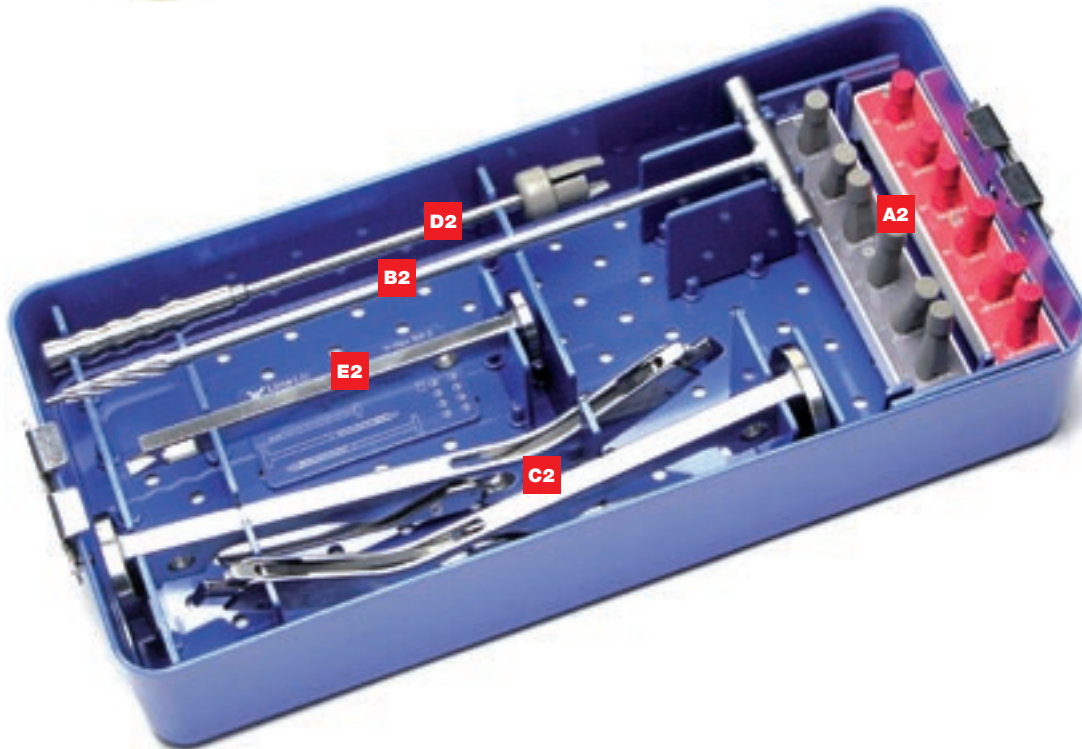
1. Pre-operative Planning

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[1]

To obtain the best results, pre-operative planning is recommended with the use of templates (showing a 15% enlarged image of the profiles). Use good quality frontal and axial view radiographs with adequate contrast that are large enough to contain the entire length of the pre-op template stems (Fig. 1).

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2. Choice of stem size

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To find the right stem size the contour lines of the proximal bulb should fill the femoral epiphyseal region in the best possible way.

Make sure that the stem does not interfere with the femoral curvature along the axial projection.

To be emphasised is that fixation of the stem is obtained at the proximal level, therefore, no support is required in the distal part.

Identify the size and type of neck to restore the correct centre of rotation, checking the antversion level in the sagittal plane.

Identify the neck resection level by making the apex of the greater trochanter coincide with the centre of the medium femoral head (Figures 2-3).

[2]

Note. Pre-operative planning provides useful indications for the correct implementation of the implant, but does not give the exact stem size to be used, which must nevertheless be checked during surgery.

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3. Position of the patient

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Note. The first figure of the surgical technique shows the posterolateral approach with the patient in lateral decubitus position. The technical procedure that follows also refers to the supine position and any type of surgical access.

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Once the fascia lata has been accessed and the external rotators sectioned, incision is made to the articular capsule and the femoral epiphysis is dislocated. At this point the femoral head is exposed.

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4. Neck resection

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After having dislocated the femur, the femoral head is resected (Fig. 5).

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5. Reaming and broaching

5. Alesatura e bocciatura

Start perforating the canal with the punch, which breaks the trochanter (Fig. 6).

The canal is then opened using a reamer (Figures 7-8).

The H-MAX system consists of a series of broaches (Fig. 9); those with an angular handle that are used in the direct lateral approach and when the patient is in a supine position, or those with a straight handle (Fig. 10) that are used in the posterior approach or when the patient is in a lateral decubitus position.

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[11]

Connect the broach to the handle by simply lifting the lever and inserting the broach with the medial part towards the lever (Fig. 11).

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[12]

Tighten the lever until it is fully closed (Fig. 12).
Start perforation of the canal using a small sized broach, rasp 1, maintaining the correct anteverision (approx. 15°C).

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[13]

Insert the broach until the teeth disappear in the diaphyseal canal. Proceed with the next size up and continue until the epiphyseal cavity is filled in the best way (Fig. 13).

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Once the best broach measurement has been achieved (which may not necessarily be the one envisaged during pre-operative planning), remove the handle and leave the broach in situ.

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6. Trial reduction

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Normally the acetabular time precedes the femoral time. Therefore, at this point a trial reduction can be performed.

Once the neck and femoral head size and type have been identified, select the trial neck that is most suitable and insert it using an appropriate instrument (Figures 14-15).

Insert the trial heads (lengths S, M or L) (Fig. 16) and perform a trial reduction (Fig. 17).

Remove the trial neck using an appropriate neck extractor.

Note. Trial heads in XL, XXL, XXXL length sizes suitable for only implants with metal heads are available upon request.

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[18]

Take an H-MAX stem of the size that corresponds to the last broach used and remove it from its sterile packaging (Fig. 18).

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[19]

Using the stem positioner, insert the stem by beating in until reaching the rasp level (Fig. 19).

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[20]

The neck can be further checked by inserting the preselected trial neck on the final stem.

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Clean and dry the tapered coupling area well before inserting the neck of the same size as the last trial reduction (Fig. 20) and perform a further trial reduction.

8. Final head insertion

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[21]

The head length can be checked once again by using the trial heads. Take a head with preset diameter and length from its sterile packaging (Fig. 21).

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[22]

Carefully clean and dry the neck taper - this is fundamental when using ceramic heads - and fix the head by pressing it in and rotating it along the neck axis (Fig. 22).

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[23]

Remove the protective cap and tap slightly along the cone axis using the femoral head beater (available upon request) (Figures 23-24).

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[24]

After having cleaned the articular surfaces, reduce the articulation (Fig. 25).

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[25]

9. Component removal

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If necessary, the prosthetic components can be removed.

To remove the femoral head, simply tap the bottom of the head in an axial direction using a punch.

Note. If only the head needs to be replaced, never use a ceramic head on the same cone; in this case the stem must also be replaced.

Therefore, remove the modular neck using an extractor with beater and remove the stem using the positioning handle (Fig. 26-27).

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For further information about our products, please visit our web site www.lima.it

This documentation has been written by Lima-Lto Sales & Technical Department

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