Revision

Modular Hip System
Sistema modulare d'anca

SURGICAL TECHNIQUE
TECNICA OPERATORIA

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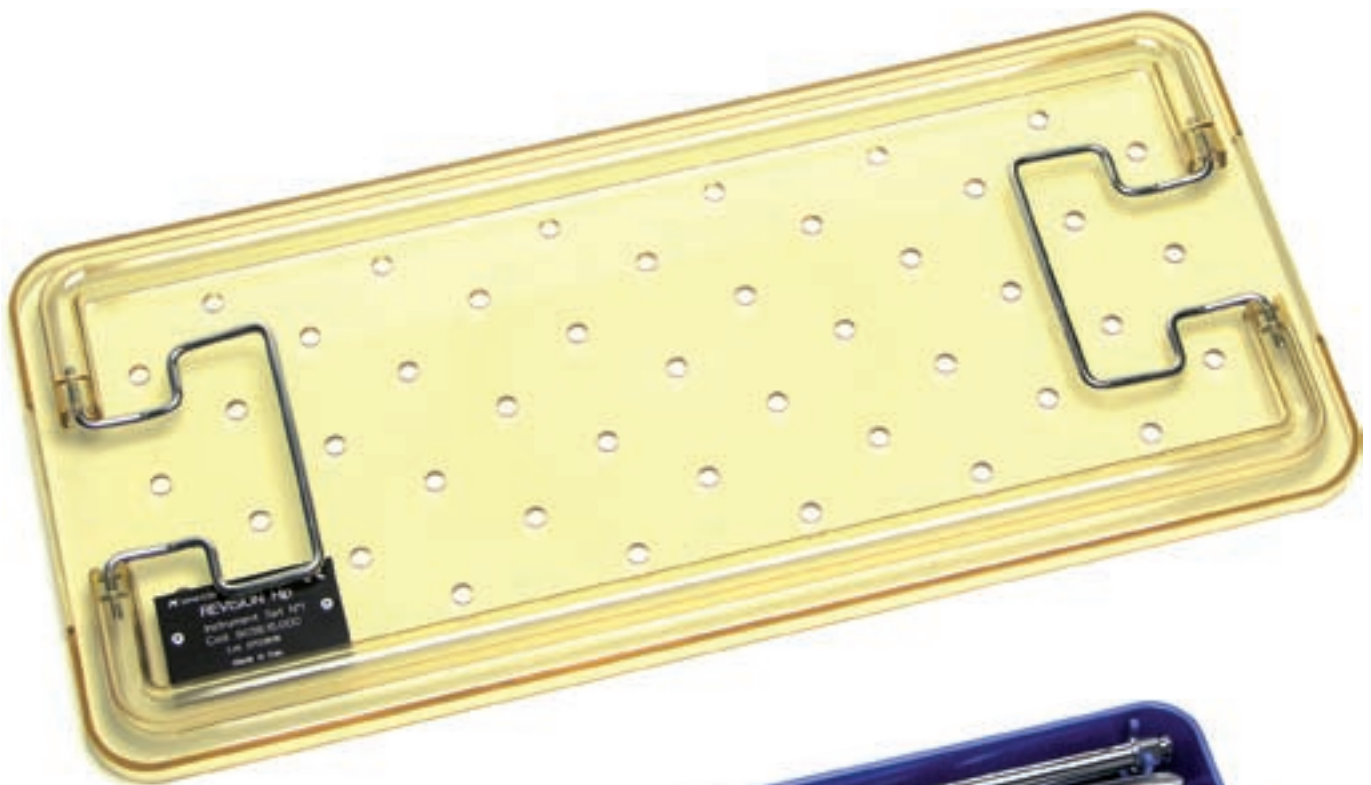
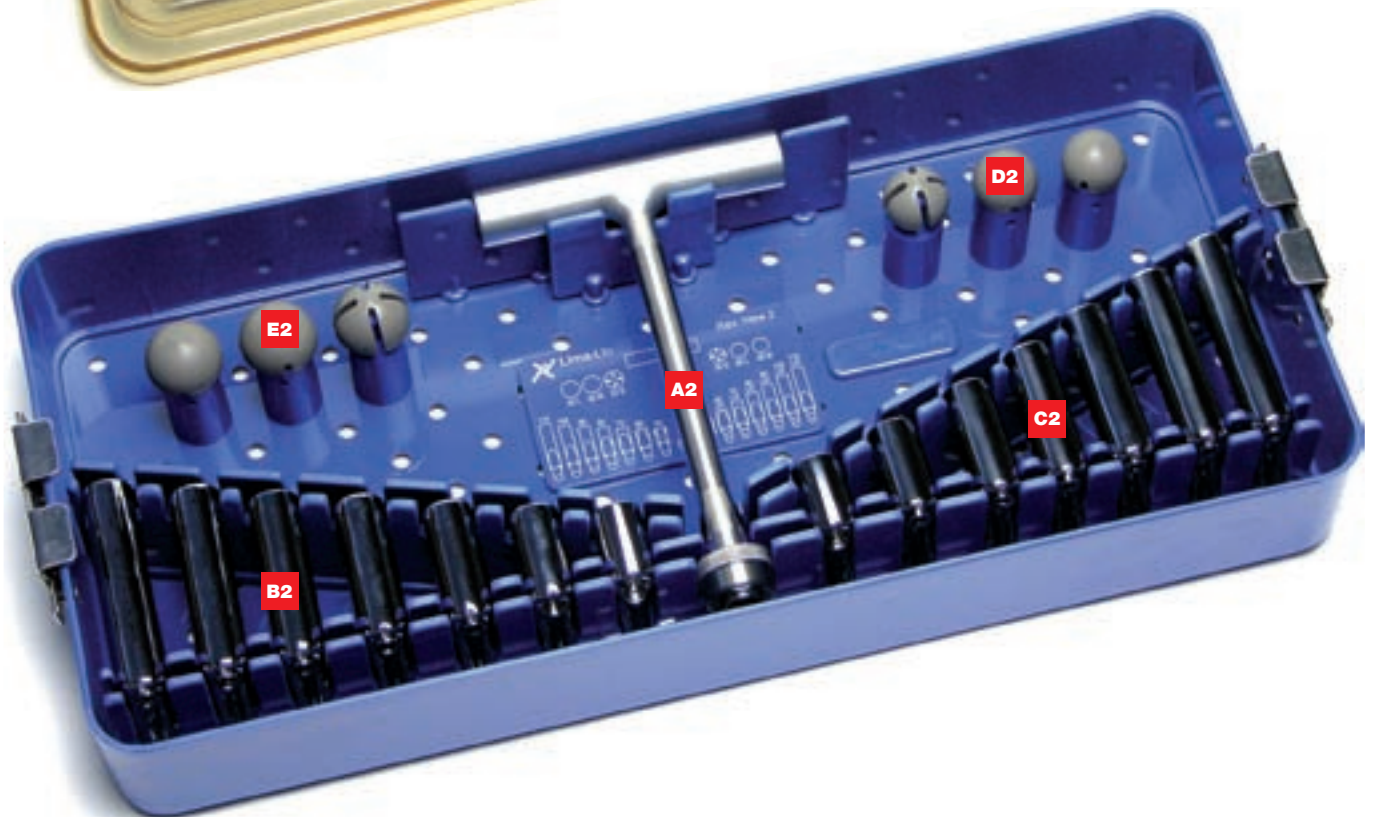
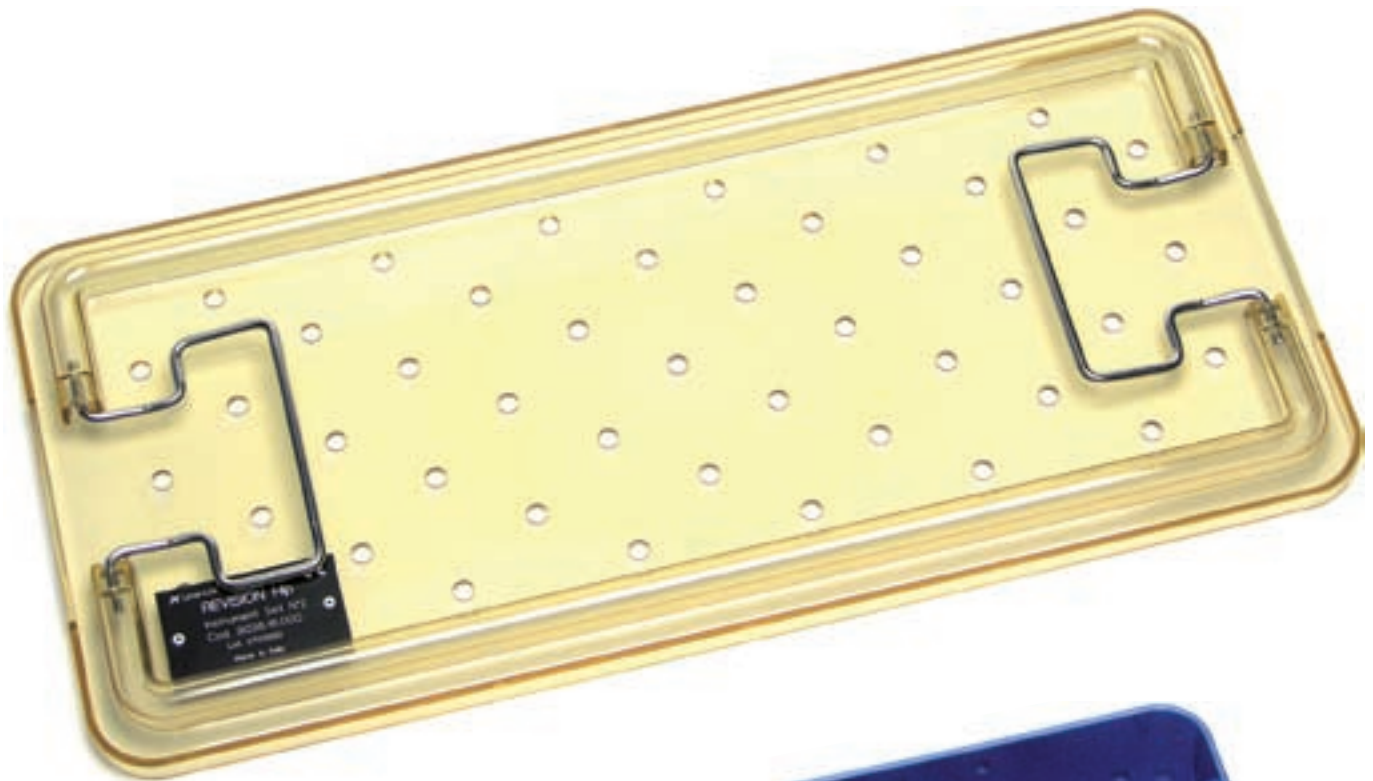
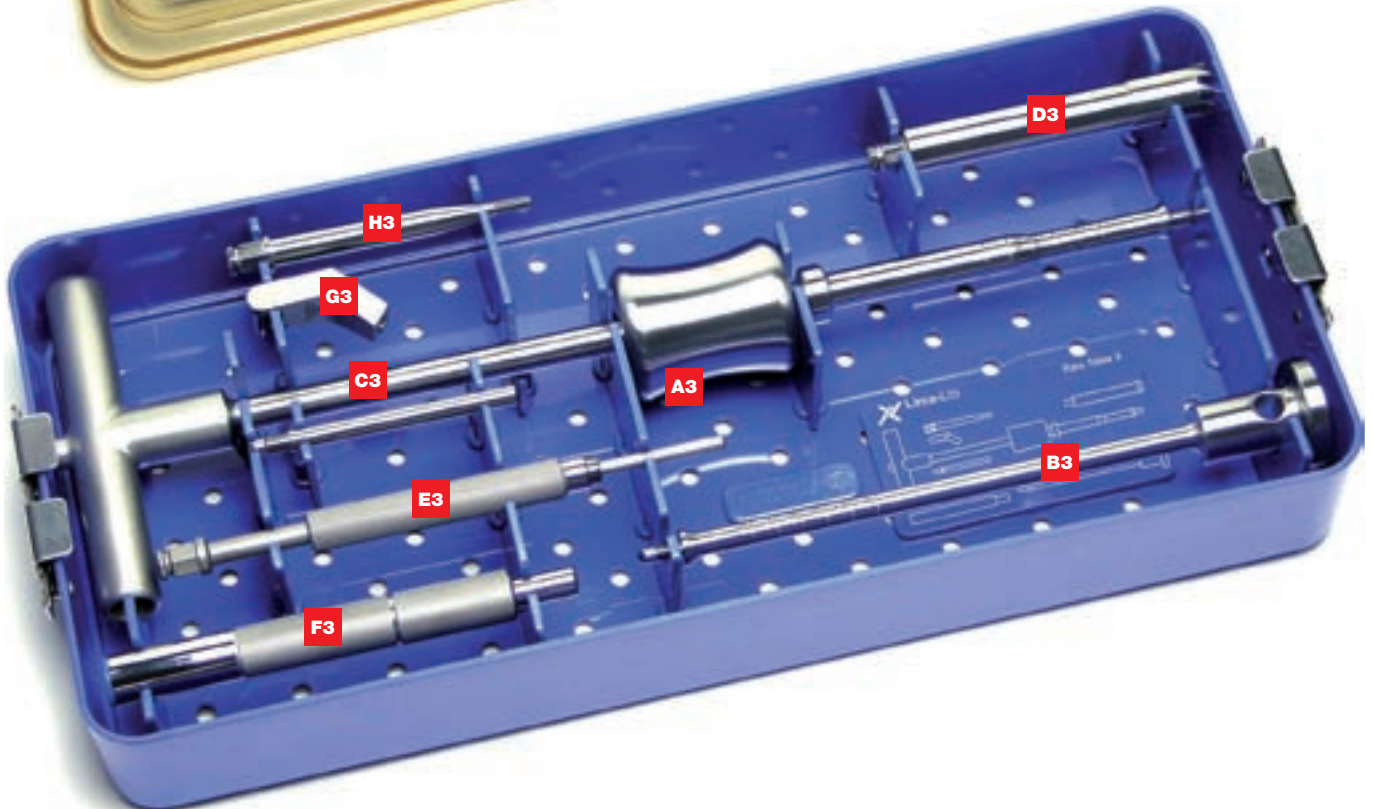
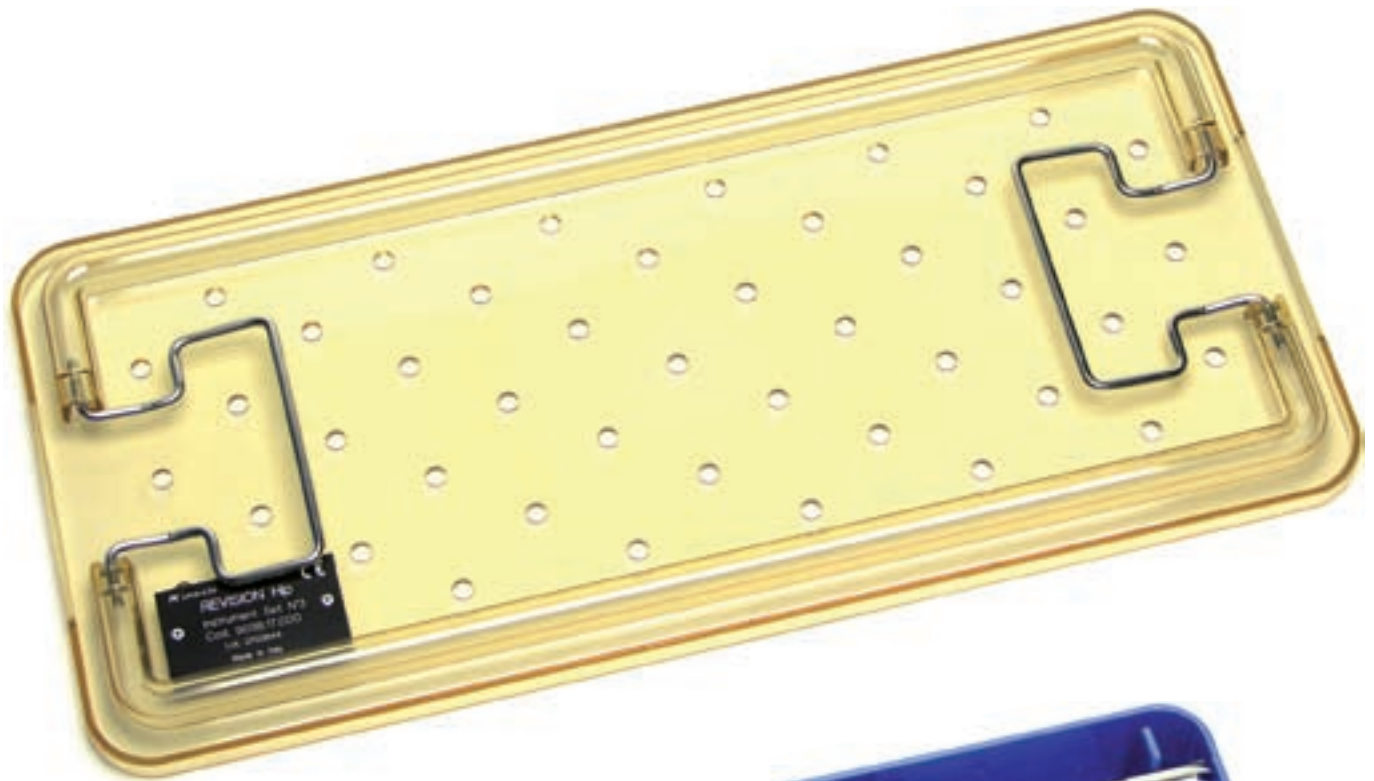
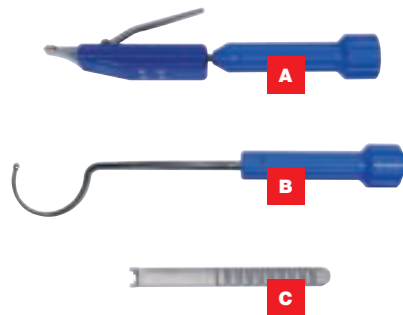
1. Pre-operative planning

Pre-operative planning is of fundamental importance for establishing the ideal length of the implant and the correct diameter of the final stem.

Stem diameter

Position the pre-surgical templates over the X-rays of the implant to be replaced, so that the fins circling the stem penetrate the endosteal cortical to a depth of about 1 mm.

Stem length

In order to accurately assess how deeply the stem descends into the femur, it is advisable for the pre-surgical planning to be performed on an X-ray of the femur also in the lateral position, so as to carefully position the distal end of the stem, and thus respect the shape of the medullary cavity along the curvature of the femur.

The distal end of the implant must fit the femur to a depth of at least 70-100 mm.

Thus for a winglet Wagner fixation the system approach reminds that the stem should be always the even size larger than the last even reamer diameter used. In any case the odd diameter reamers are suggested to a minimal diaphyseal canal caliper adaptation when necessary.

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1

Neck height and offset

Once the template of the stem is properly positioned, using a frontal X-ray select the appropriate neck size by lining up the tip of the greater trochanter with the center of the femur head.

When choosing the neck, bear in mind that it may be necessary to modify the lever arm offset and the length of the limb.

It goes without saying that decisions made in the planning stage are always subject to modification during surgery.

If using the trans-femoral approach, it is advisable to accurately measure the length of the stem to be removed (as a rule, the X-ray images are magnified by 13-15%, so after measuring the length of the stem, divide the value by 1.13-1.15).

This measurement will be useful for establishing how long to make the diaphysectomy, if required (Fig. 1).

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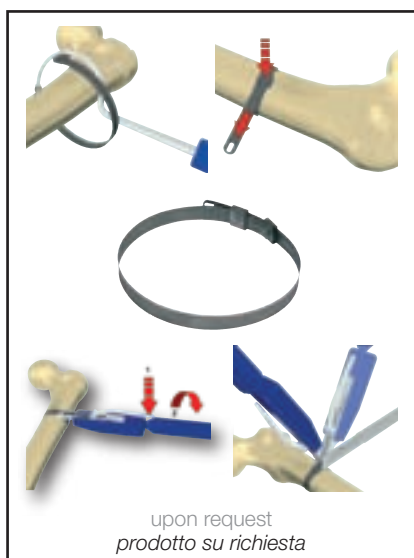
2. Reaming the femoral canal

2. Alesatura del canale femorale

After removing the old stem, if diaphysectomy of the femur has been performed, a precautionary reinforcement cerclage around the femoral diaphysis can be put to avoid accidental bone splitting.

The LimaLto cerclage system is available upon request

- **cerclage band** (cod. 8610.28.100) made of pure Titanium, grade 4
- **band tightener** (cod. 9086.10.100)
- **positioner** (cod. 9086.10.200)
- **cutter/bender** (cod. 9086.10.300)

Start with a small diameter **Reamer** (code 9038.10.010...) to bore through bone plugs, smooth out any unevenness and avoid wrong directions.

Switch to progressively larger diameter reamers until actually scraping the bony walls.

If possible, a radiographic monitoring is suggested.

In case of trans-femoral approach, the reamers must bore a deep enough hole to give the stem a distal fit of at least 7-10 cm beyond the window. Therefore, the distal end of the reamers should correspond to the anatomical curve of the femoral procurvation (Fig. 2).

[2]

This is the ideal position for gauging

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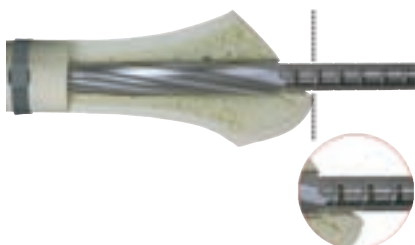
the size of the stem and modular neck to be used in the final implant.

As far as the stem diameter is concerned, routinely the choice would get for an even size larger than the last even reamer diameter used, for instance, 16 mm reamer and 18 mm stem.

When the bone trophism can obstruct the wing penetration it is nonetheless recommended the use of odd diameter reamers to refine canal preparation. Thus considering the example of size 18mm of the stem the odd reamer 17mm can be used to adjust the femoral canal caliper.

[3]

To select the correct stem and neck size, refer to the gauge on the reamer axis at the level of the greater trochanter (Fig. 3), where two numbers are marked.

The first refer to the length of the stem, the second to the height of the neck.

For example, if the gauges read "200 + 80", the stem to be used in this case is the 200 mm size, with an 80 mm trial neck.

The reamer axis also provides a useful indication of how to make the best use of the implant's 4° angle.

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If using the trans-femoral approach, the reamer axis generally appears anterior to the ideal neck axis (Fig. 4), so that the stem, (usually long) will be fitted with the tapered portion tilted posteriorly, thus restoring the normal neck axis.

If, conversely, the trans-femoral approach is not employed, the reamer axis generally appears posterior to the ideal neck axis, therefore in this case the stem (usually short) will be fitted with the tapered portion tilted anteriorly (Fig. 5).

It is of course also possible for the stem to be placed with the neck in a varus or valgus position.

The surgeon must make this decision as circumstances dictate, by comparing the reamer axis with the meta-epiphyseal profile of the femur.

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cod. 9038.10.110

[6]

Take out from the sterile package the Stem whose diameter is one size larger than the last reamer diameter used (for instance, 16 mm reamer, 18 mm stem).

Two different impactors may be used to tap the stem in the diaphyseal shaft: a **Dynamic Impactor** (code 9038.10.100) and a **Manual Impactor used with a hammer** (code 9038.10.110) (Fig. 6).

Screw the manual impactor onto the stem.

Four grooves at the base of cone are reference of the possible stem position in according to the femoral procurving and antecurving or varus and valgus of neck, as already described (Figs. 4 e 5)

Push the stem into the canal. Definitely strike down gently tapping the stem until distal fixation is obtained (Fig. 7).

The impactors feature a gauge to determine how deep down the stem has to be driven (Fig. 8).

The gauge indicates the neck sizes.

Taking the tip of the greater trochanter as a reference, drive the stem down until it reaches the neck height selected during the reaming stage (80 mm in our instance).

If difficult is too much to get the stem

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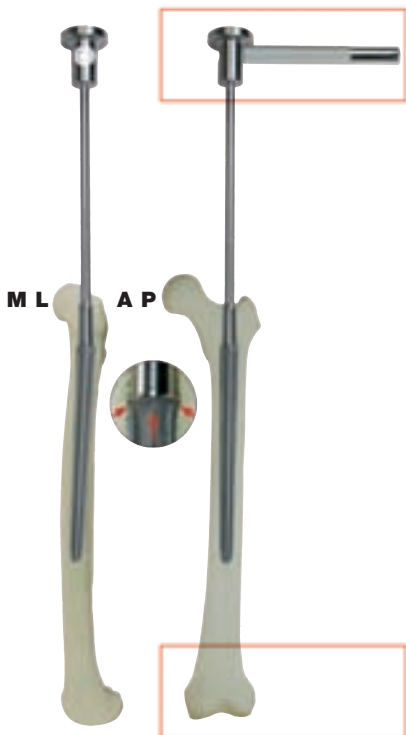
[7]

[8]

in, stop short of the preset depth (provided that a minimum neck height was not chosen, i. e. 500 mm). Nevertheless odd diameter reamers refine canal preparation to adjust for a stem descending (thus considering the example of size 18mm of the stem, if it were necessary, the odd reamer 17mm can be used to adjust the femoral canal caliper).

Conversely, if the stem reaches down the pre-established level but does not give you a feeling of good fit, push it further down until proper upper level is reached (provided that a maximum height neck was not chosen, i. e. 110 mm).

If the reamer diameter chosen was not correct – or in rare cases of poor bone trophism – you may even find it all too easy to get the stem down.

If this is the case, we suggest using a larger diameter stem and special care when inserting it.

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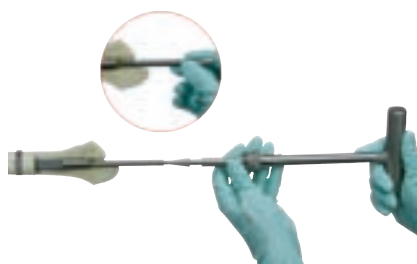
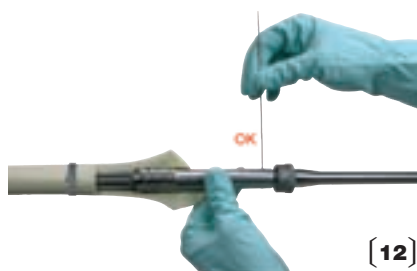
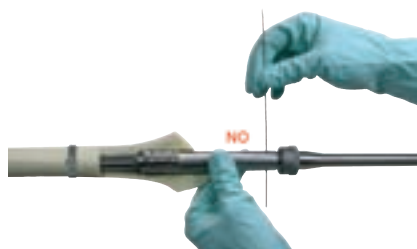
[11]

[12]

To prepare the femoral neck seating, use the **Neck Reamer** (code 9038.10.120) for proximal reaming.

Screw the **Neck Reamer Guide** (code 9038.10.115) tight into the stem Morse-type taper using the **Allen Wrench** (code 9095.10.117) fitted in the **Manual Snap Wrench** (code 9095.10.110) (Fig. 9).

Fix the manual snap wrench onto the neck reamer and insert the reamer in the guide (Figs. 10 and 11).

Rotate the reamer into the femur meta-epiphyseal cavity until end of stroke.

Note The inspection hole on the reamer allows by a metallic wire or a needle to check reaming is completed (Fig.12).

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5. Trial reductions

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[13]

After accurately washing the stem taper to remove any bone debris left over from the previous stage, fit the **Trial Neck** on it (code 9038.10.160...) choosing the size that was read previously on the **Stem Impactor gauge** (Fig. 13) and in standard 135° configuration.

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[14]

Carefully select the neck anteversion; then tighten the neck locking screw using the Allen wrench (Fig. 14).

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[15]

This stage does not usually require using the **Neck Stopper** (code 9038.10.240), although you may decide to use it anyway for more safety reasons.

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If the implant length does not fit any trial head, replace the trial neck and pick up a new one of a different height.

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Since the neck height increments are of one centimetre each, one can easily figure out the correct implant size.

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If the height were still to be excessive when using a minimum size Trial Neck (50 mm), drive the stem a bit further down. If the stem would not descend enough, remove it and provide for a canal caliper refining by the larger odd

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[16]

reamer, than re-insert the stem and check for the height level.

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[17]

If the standard neck offset does not fit for a proper articular ratio, replace it choosing the lateralizing 131° one.

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Note (test according to ISO 7206-4):

The lateralized modular necks can't be coupled with 14 mm diameter stems. All the modular necks can be coupled only with modular femoral heads of length not over "L".

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Once the right size of the head and neck has been selected, remove the trial neck unscrewing the locking screw.

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If it were too hard to get the trial neck out, use the **Trial Neck Extractor** (code 9038.10.250) which is screwed (Fig. 16) on the stem impactor where the neck stopper may be fitted (Fig. 17).

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6. Fitting the final neck

6. Innesto del collo definitivo

Take the final Neck out of the sterile package of the size selected in the previous stage. (Caution! The neck package also contains the locking screw).

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Screw the **Neck Impactor-Extractor** (code 9038.10.230) in the neck threading after removing the long coaxial screw (which is only meant for neck removal) (Fig. 18).

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[18]

After accurately washing and drying the stem taper, insert the neck on the stem Morse-type taper with the anteversion selected previously.

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[19]

Gently tap the impactor along its axis using a hammer (possibly a plastic one) in order to fit the two cones (Fig. 19).

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Finally, unscrew the impactor off the neck.

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[20]

Now, insert the locking screw in the neck hole and tighten it using the Allen wrench placed in the manual snap wrench (Fig. 20).

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To firmly tighten the screw in, insert the neck stopper in the neck taper (Fig. 21).

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[21]

The combined actions of the screw fastening wrench and of the neck stopper keep any torque load from being transmitted to the femur.

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If a bone splint was overturned for trans-femoral approach, position it back into its original site in order to

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[22]

close the femoral window and reinforce it with metal cerclage, tacking care of avoiding the direct contact between the wire and the prosthesis.

Finally, fit the selected final femur head on the neck taper (clean and dry) and proceed to implant's final reduction (Fig. 22)

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7. Removing the neck

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Should it need to remove the femoral neck, proceed as follows:

Unscrew the locking screw (using the neck stopper)
Screw the neck impactor-extractor tightly on the neck and in it the long screw which was removed previously.

After a long stroke, the rod thus inserted will get in contact with the stem.

Keep the neck stopper well in position and continue to drive the screw in with the Allen wrench (Fig. 23).

An immediate a reduction of the load transmitted can be felt by the Allen wrench.
At this point, both stem and neck are disassembled.

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Clinical Cases

Casi clinici

pre-op 75 years old, female, peripros-
thetic fracture

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6 years post-op

post-operatorio a 6 anni

For further information about our products, please visit our web site www.lima.it

This documentation has been written by Lima-Lto Sales & Technical Department

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X Lima Deutschland GmbH