Delta

Acetabular Cups
Coppe acetabolari

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History and Philosophy

Storia e filosofia

The Delta system was designed in 1999 in cooperation with Prof. Giacometti Ceroni (Galeazzi Institute - Milan, Italy).

The objective of this system is that of enhancing the hip joint stability by reducing the risk of dislocation in hip arthroprosthesis, and the tribology performance of articulating components.

On the basis of Charnley's experience, the use of small diameter artificial articulations for metal-polyethylene, ceramic-polyethylene couplings, can be correlated to the reduction in torque transmitted by friction between the femoral and the acetabular component and hence also by the reduced volumetric wear.

The long term use of small femoral head has brought at the evidence another factor as well able to induce deteriorating phenomena for the polyethylene liners. The more the spherical area decreases the more the contact stress is less distributed, that accentuating the penetrating effect in case of couplings requiring the use of polyethylene.

The diameter of the articulation plays also important leading role in defining the range of the joint motion.

A large diameter coupling, for example 36mm, offers considerable advantages in terms of safety.

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ceramic-ceramic couplings involves torques weighing on the acetabular perimeter that are inferior or equal to those exerted by 28 mm couplings, thus assuring a higher articular junction stability with a lower dislocation risk, since:

► in large diameters the increase in the articular range of motion reduces the danger of contact between the femoral neck and the acetabular rim.

► the increase of the typical coupling radius increases the distance the femoral neck would need to cover in order to come out of the acetabular seat.

The Delta system was devised keeping these objective facts in mind. The system also includes the two most common fixation concepts (press-fit and screw), that have been revised and improved to offer suitable surgery performance (a better control of the implant position, an easy insertion of articular liner, a better initial stability, ergonomic instrumentation) as well as clinical performance (bone saving, integration).

The Delta system also offers the most advanced orthopaedic technology in terms of design, with a new liner/metal back cone coupling section, and of materials, with the implementation of the innovative BioloX® Delta ceramic as well as of a new cross-linked polyethylene for which Lima-Lto has studied and approved a specific manufacturing process.

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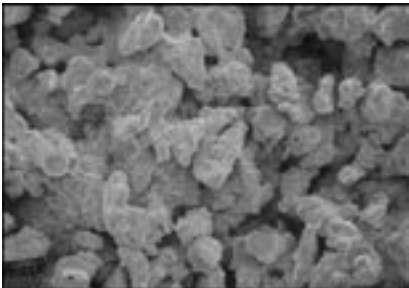
Porous Titanium Coating
Rivestimento di Titanio Poroso

Bone in-growth on Porous Coating
Integrazione ossea su rivestimento poroso

► The shell of Delta acetabular cups is manufactured with a Titanium alloy, Ti6Al4V.

► The surface finishing is available in various textures: macro-roughened, simple coating of 170 µm minimum thick porous titanium process, as well as double one characterised by an additional 60 µm layer of hydroxyapatite.

The coating is deposited by means of a Plasma Spray

► The liners are available in BioloX® Forte ceramic or in the innovative BioloX® Delta ceramic, in 'MET' special CoCrMo alloy (NB: specific heads for coupling), in either the traditional UHMWPE or in the cross-linked UHMWPE X-LIMA version, both ringed by a Titanium alloy, Ti6Al4V band.

The Titanium alloy, Ti6Al4V features also both the types of polyethylene liners, because it is used to realize the elastic band and the polar peg coat.

► Femoral heads are supplied in BioloX® Forte, offering a complete coupling compatibility with BioloX® Delta ceramic, in 'MET' special CoCrMo alloy.

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
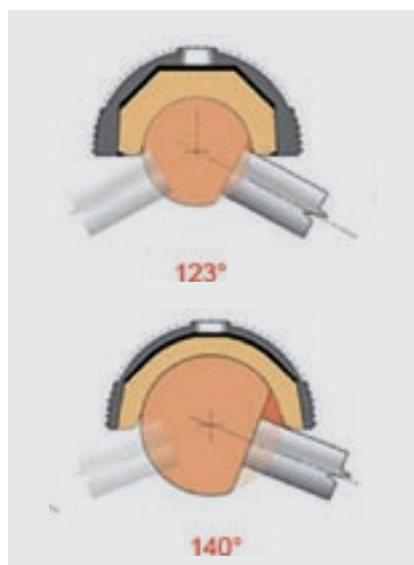
Biomechanical Characteristics

Caratteristiche biomeccaniche


The use of the 36 mm diameter, that is available in the Delta system starting from the 50 mm size shell, allows the articulation a range of motion exceeding 140°, with respect to 123° of the coupling that can be obtained with a 28 mm diameter. This reduces the risk of accidental contacts between the femoral neck and the metal shell rim.

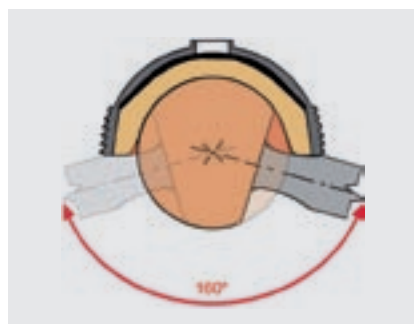If a Delta system acetabular cup is combined with a Lima-Lto femoral stem with tapered neck section, the articular joint thus repaired can reach a range of motion close to 160° (maximum value with 36 mm diameter coupling).

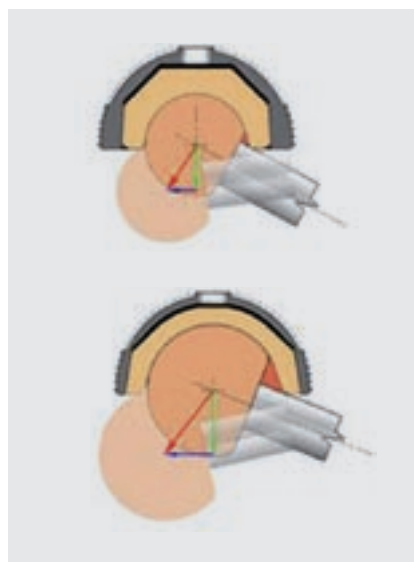
With larger diameters, the femoral head must obviously cover more distance in order to come out of its seat, thus assuring a reduction of dislocation risk in case of abnormal tissue laxity or of reasons that determine an incorrect maintenance of joint motions.

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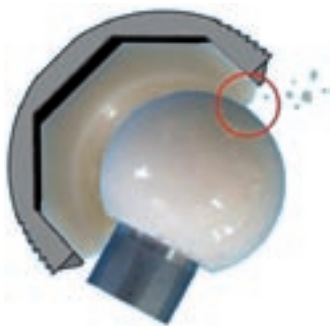As a result, risks of a lever effect with loss of congruence and of reduction of the area of head/liner contact are reduced, these being phenomena that involve a load concentration in one contact point only with consequent ceramic breakage.

The Delta system combines the concept of a large coupling with the concept of two main primary fixations: press-fit and screw-in.

It offers a complete option of surgical solutions, since the first implant, to the CDH treatment and the early acetabular revision, in the way, the choice of the implant is based on the philosophy that is more suited to the bone morphology and trophism.

Equipped with equatorial retention grooves, the PF model is diametrically oversized by 2 mm with respect to the corresponding acetabular milling cutter. This way the mechanical interference between the cotyle and the acetabulum allows an optimal press-fit effect.

Three holes in the cranial area equipped with removable plugs, allow a further stabilisation by means of bone screws.

The FINS cup model, 2 mm press-fit and cranial holes with removable plugs as well set, features a design highly functional to an acetabular wall whereas a suitable surgical morphology stands, but for which a new ex-novo site needs, for some specific cases of early revision or osteotomies outcome, and in the total joint reconstruction of femoral neck fracture.

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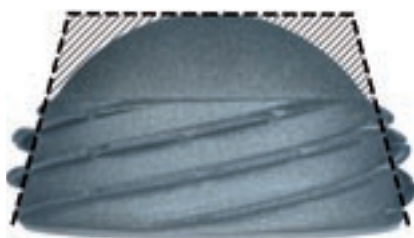
For all those cases, equatorial torsion stability can be bad compromised, and a circumference grooves equatorial design cannot be anymore effective. Therefore the FINS cup is equipped by longitudinal equatorial fins. Moreover the hemisphere is more congruent to the local area, making possible an increase of mechanical friction. All those peculiar technical hints arise the general torsion stability of the implant.

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Delta ST

In order to generate an adequate mechanical friction with the bone wall and allow the best biological integration possible, the PF and FINS cups have a porous contact surface available in double coating. An external layer of hydroxyapatite is deposited to increase stimulation of bone in-growth into a sub-layer of porous titanium.

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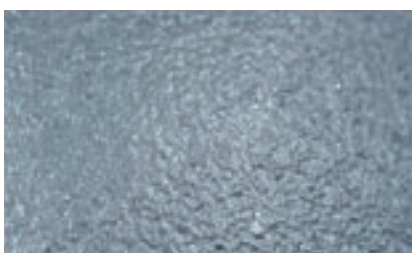
Delta STC

The ST and ST-C models are provided with six helicoids, the section of which has been designed to guarantee an optimal initial stability and to support torsion loads weighing in the equator area, without however interrupting the continuity in the bone wall. The hemispheric-shaped shell design allows a complete support in the acetabular seat. Therefore the load is transferred to the entire surface of the subchondral bone, without generating over-loads in the proximity of the helicoids, and allowing a considerable bone saving. In fact, with respect to the equivalent troncoconical section for 50 mm size, it is possible to maintain up to 5 cm³ of bone.

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Ti6Al4V with Porous Titanium Coating
Ti6Al4V con rivestimento di Titanio poroso

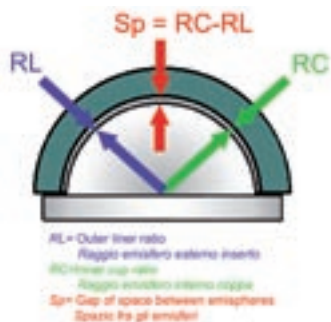
The high congruence between shell and bone of hemispheric acetabular cups with sections that can be screwed in is well-known.

Therefore the ST model is offered in a classical version for trophic bone structure without coating and the ST-C model is offered in a single porous titanium layer to further enhance the mechanical friction with the acetabular wall and stimulate integration for poor and atrophic bone.

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Hemispherical Coupling
Accoppiamento emisferico

Product Characteristics

The Delta system has four lines of metal shells that can be combined in turn with seven types of inserts, made of different materials and specific articular seat geometries.

The coupling between liner and metal back is of a tapered type, a design that Lima-Lto has been using since 1978 for the following reasons:

- ▶ the articular load allows a 100% continuous and non-deformable contact
- ▶ mechanical constraint is proportional to the bearing load

In fact for liners of semispherical design and in which the connection to the cup is provided at the equator

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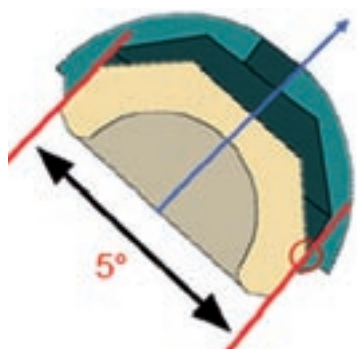
level by self-retention elastic ring or snapping device, the unavoidable gap of space of constructive origin due to the different curving ratios is source of micromovements.

Intra-surface repeated contact can lead to polyethylene debris

Thanks to the extensive experience gained by Lima-Lto, the Delta system offers a new liner design, developed in cooperation with CeramTec AG.

All the types of conical coupling are often deemed not to be easy de-coupling. In the first instance, that can be possible when the taper is minor than 10° circa, as the 'euro-taper' for the stems which is of 5°40', and in peculiar conditions of friction.

In the past ceramic liners for acetabular cups were always produced with a taper identical to the stems one. When the coupling is of direct contact, ceramic on metal back, wrong alignment could lead to unfortunate effects, like rim chipping. Moreover in case of revision, disassembling this design often makes removal extremely difficult.

In the Delta Acetabular System the taper is quite close to 19°.

CeramTec AG company has made the choice for such an angle to decrease risk of rim chipping, favouring the ceramic liner inserting manoeuvre and the de-coupling in case of removal,

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Neutral liner
Inserto neutro

Protruded liner
Inserto protruso

and this last feature is as much feasible also for metal and polyethylene ones.

The liner manufacturing material does not affect the connection which is stable and safe.

Nevertheless for some cases the polyethylene liner can couple into the metal back with difficulty during the surgery.

To solve the problem, Lima-Lto has set up an ingenious as well as a simple system (patent pending).

Near to the area of contact with the inner conical lodge of the acetabular cup of the Delta System, a 0,7 mm thin elastic band made of Titanium alloy rings the polyethylene liner.

The band outlines both internally and externally the liner conicity. The mutual immobility between cup and band and between band and liner is based on the conical fixation: the bigger the axial bearing load the greater is the stability of the surfaces in contact.

The band is in any case appropriately manufactured in such a way to compress the liner circumferentially, since before it is inserted into the cup lodge. Once the liner is inserted and axially beaten, the liner / metal back coupling is extremely stable also after bearing load released.

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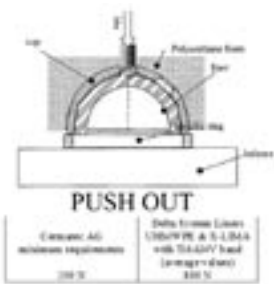
As the greatest guarantee for safety, all the liners are in compliance to the Ceramtec AG requirements for ceramic liners.

This method features also two safety mechanism for the liner handling.

- The inferior rim of the band penetrates the liner to guarantee the whole liner-band extraction when removal is required.

- Two locks are engraved 180° oppositely onto the conic mantle of the liner, thus assuring rotational swinging absence between liner and band.

All the polyethylene liners of the Delta series are equipped by a small polar peg. A thin of Titanium alloy coats it. The small peg performs two functions:

- Driving the insertion maneuver
- Closing the polar hole of the acetabular cups, to preserve from biological re-flux the interior.

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Delta PF Cup
Coppa Delta PF

The rim of the metal shells is provided with a bright surface finish to avoid possible abrasions due to accidental contact with the femoral neck

In the PF and FINS models, in addition to the three cranial holes, a further polar hole, also equipped with removable plug, allows fastening to the positioning device (NB: to allow a fast hanging

Delta FINS Cup
Coppa Delta FINS

Delta ST Cup
Coppa Delta ST

Delta ST-C Cup
Coppa Delta ST-C

onto the inserting wrench the cups are available without the polar plug, which is still included into the packaging of the ceramic and metal liners) The special design, implemented internally on the embedded polar bottom, allows a solid grasp of the wrench device and consequently the possibility of turning the shell in both directions with respect to the acetabular bone to adjust its position, without any risk of disengaging.

The ST and ST-C versions are equipped with the polar hole, to which the insertion key is fixed. A profile similar to that of the PF version embedded on the polar bottom makes it possible to maintain a solid fastening, and therefore to screw in the cup. (NB: also for this cups the polar plug is still included into the packaging of the ceramic and metal liners)

Helicoidal profiles allow for an acceptable direction stability during the screwing manoeuvre, thus reducing the risk of covering and antiversion errors.

In the ST-C cups two holes of inspection allow a fine adjustment during the insertion as per indication of a peculiar use.

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Indications

Indicazioni

Cementless acetabular system with biological fixation for total hip replacement, where the initial stability of the acetabular bone metal shell is guaranteed by the mechanical anchoring on the equatorial portion thanks to the press-fit effect in the PF and FINS versions or to the screwing in for the ST and ST-C versions.

Biological fixation can be performed by means of high-roughness external surface finishing, available in different textures.

Ideal for coxarthrosis treatment for CDH treatments and for the early revisions, whereas a bone structure of adequate resistance stands.

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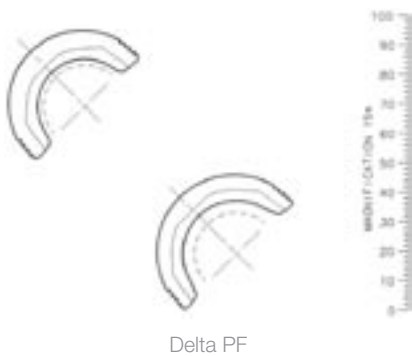
Sizes

Taglie

For each metal shell line, i.e. PF and FINS cup with Porous Titanium double coated with HA, ST cup without coating or ST-C with porous titanium single coating, the system is available in ten sizes with 2 mm intervals between each one, compatible with specific liner sizes produced in different materials:

- ▶ shell size 42 mm for the FINS model, that can be matched with liner size S (not ceramic)

- ▶ shell sizes 44, 46, 48 mm, that can be matched with liners of size S

- ▶ shell sizes 50, 52 mm, for the PF model, that can be matched with liner size M, suggested for BIOLOX® ceramic coupling

- ▶ shell sizes 50, 52 mm, for the PF model, that can be matched with liner size L (not in ceramic)

- ▶ shell sizes 54, 56, 58, 60, 62 mm, that can be matched with the liners of size L

The following inserts:

- ▶ UHMWPE size S, M and L for 28 mm diameter heads in neutral version or with anti-dislocation rim (20° protrusion)

- ▶ cross-linked X-LIMA UHMWPE, size S for 28 mm heads and size M for 28 mm and 32 mm diameter,

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L for 32 mm and 36 mm in neutral version or with anti-dislocation rim (20° protrusion)

► BIOLOX® Delta, size S for 32 mm heads in BIOLOX® Delta or BIOLOX® Forte ceramic, and sizes M and L for 36 mm heads in BIOLOX® Delta or BIOLOX® Forte ceramic

► BIOLOX® Forte, size S for 32 mm heads in BIOLOX® Delta or BIOLOX® Forte ceramic, size M and L for 36 mm heads in BIOLOX® Delta or BIOLOX® Forte ceramic

► "MET", special CoCrMo alloy, sizes S, M, L for 36 mm "MET" heads

NOTA: polar plug for the cup is available within the packaging of Ceramic and Metal Liners

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Product Codes

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Delta PF Cup
Coppa Delta PF

Coppe a Acetabular Cups Delta PF Coppe acetabolari Delta PF acetabolari

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Nota: Coppe dia. 50 e 52 mm Large non abbinabili a inserti Biolox
Note: Cups dia. 50 and 52 mm Size Large not suitable for Biolox Liners

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Delta ST-C Cup
Coppa Delta ST-C

Delta ST Cup
Coppa Delta ST

Acetabular Cups Delta ST-C Coppe acetabolari Delta ST-C

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Acetabular Cups Delta ST Coppe acetabolari Delta ST

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Nota: Coppe dia. 50 e 52 mm Large non abbinabili a inserti Biolox
 Note: Cups dia. 50 and 52 mm Size Large not suitable for Biolox Liners

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UHMWPE X-LIMA + Ti6Al4V

	size taglia	for heads dia. per teste dia.
5885.51.055	S	28mm
5885.51.058	M	28mm
5885.51.158	M	32mm
5885.51.060	L	28mm
5885.51.160	L	32mm
5885.51.260	L	36mm

Protruded Liners for Heads Dia. 28 mm, Dia. 32 mm and Dia. 36 mm
Inserti protrusi per teste dia. 28 mm, dia. 32 mm e dia. 36 mm

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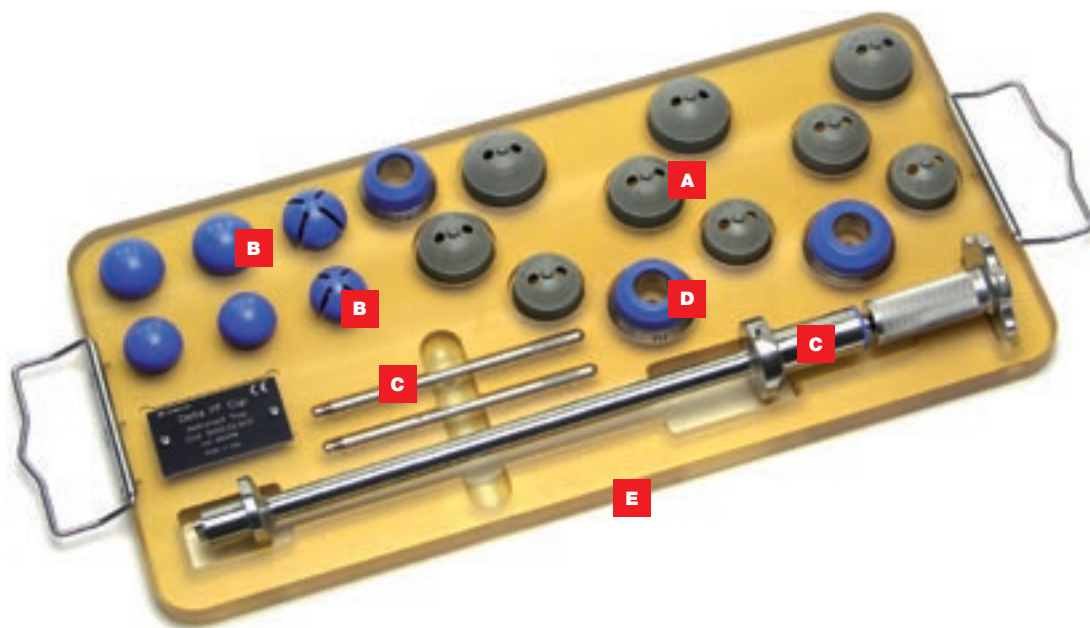
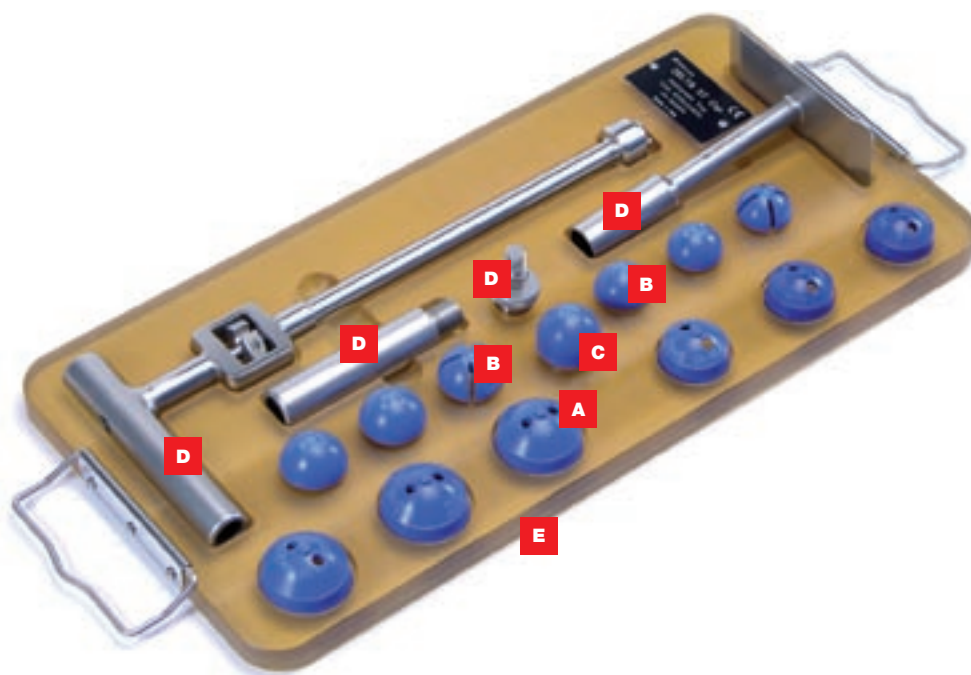
ref.		description	descrizione	qt.
9055.50.020	A	Trial Liner, Large for Head Dia. 28mm	inserto prova Large per teste Dia. 28mm	1
9055.50.025	A	Trial Liner, Large for Head Dia. 32mm	inserto prova Large per teste Dia. 32mm	1
9055.50.030	A	Trial Liner, Large for Head Dia. 36mm	inserto prova Large per teste Dia. 36mm	1
9055.50.220	A	Trial Liner, Small for Head Dia. 28mm	inserto prova Small per teste Dia. 28mm	1
9055.50.225	A	Trial Liner, Small for Head Dia. 32mm	inserto prova Small per teste Dia. 32mm	1
9055.50.230	A	Trial Liner, Small for Head Dia. 36mm	inserto prova Small per teste Dia. 36mm	1
9095.10.521	B	Trial Head Low Taper 12/14 Dia. 32mm Short	testa prova cono ribas. 12/14 Dia. 32mm Short	1
9095.10.522	B	Trial Head Low Taper 12/14 Dia. 32mm Medium	testa prova cono ribas. 12/14 Dia. 32mm Medium	1
9095.10.523	B	Trial Head Low Taper 12/14 Dia. 32mm Long	testa prova cono ribas. 12/14 Dia. 32mm Long	1
9095.10.531	B	Trial Head Low Taper 12/14 Dia. 36mm Short	testa prova cono ribas. 12/14 Dia. 36mm Short	1
9095.10.532	B	Trial Head Low Taper 12/14 Dia. 36mm Medium	testa prova cono ribas. 12/14 Dia. 36mm Medium	1
9095.10.533	B	Trial Head Low Taper 12/14 Dia. 36mm Long	testa prova cono ribas. 12/14 Dia. 36mm Long	1
9057.20.320	C	Liner Impactor for Dia. 36mm Head	impattatore inserto per testa Dia. 36mm	1
9055.21.010	D	Wrench for Cups DELTA-ST	chiave inserimento coppa DELTA-ST	1
9055.21.900	E	Instrument Tray	vassoio strumentario	1
9095.10.920		Sterilizable Box	box sterilizzabile	1

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The Development of Large Ceramic Heads to Obtain More Stable THA with Wider Range of Motion

R. Giacometti-Ceroni - P. Dalla Pria

Proceedings 6th International BIOLOX® Symposium March 23/24 2001

Prosthetic surgery in degenerative hip diseases. Materials: perspectives and prospects. Ceramic.

P. Dalla Pria - W. Burger - L. Giorgini

G.I.O.T. 2002; 28(Suppl. 1):S319-25

Alumina-on-Alumina Coupling with 36 mm Heads

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The evolution of the Use of Ceramics in the Hip Joints

P. Dalla Pria

S.I.O.T. 2002 - Lido di Venezia, Italy - 22/23 October 2002

A 36 mm Diameter Coupling: Conceiving and History

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L. Zagra - R. Giacometti Ceroni

S.I.O.T. 2002 - Lido di Venezia, Italy - 22/23 October 2002

The Evolution of the Use of Ceramics in the Hip Joints

P. Dalla Pria

Aussies & Kiwis in Italy 2003 - Lima-Lto S.p.A., Villanova di San Daniele (Udine), Italy
- 19 May 2003

- Rationale for Dia. 36 mm Heads in Ceramic-Ceramic Joint
R. Giacometti Ceroni - L. Zagra - P. Dalla Pria
Aussies & Kiwis in Italy 2003 - Ospedale Santo Spirito, Rome, Italy - 21 May 2003
- Ceramic-Ceramic Coupling with Big Heads. Clinical Outcome.
R. Giacometti Ceroni - L. Zagra
A.A.O.S. 2004 Annual Meeting - San Francisco (California), USA - 10/14 March 2004
- Ceramic on Ceramic Coupling with Big Diameter Heads: Clinical Evaluation of the Dislocation Rate.
L. Zagra - R. Giacometti Ceroni
Congresso SMC - Seoul, South Korea - 5 June 2004
- The Evolution of Ceramic Coupling: from 28 to 36 mm
F. Macchi
7° E.F.O.R.T. European Congress 2005 - Lisboa, Portugal - 4/7 June 2005
- Ceramic-Ceramic Coupling with Big Heads. Clinical Outcome.
R. Giacometti Ceroni - L. Zagra
7° E.F.O.R.T. European Congress 2005 - Lisboa, Portugal - 4/7 June 2005
- The Evolution of Metal on Metal Coupling: History and New Trends.
A. Piotrowski - G. Bensmann
7° E.F.O.R.T. European Congress 2005 - Lisboa, Portugal - 4/7 June 2005
- Metal on Metal Coupling with Big Heads.
F. S. Santori - N. Santori
7° E.F.O.R.T. European Congress 2005 - Lisboa, Portugal - 4/7 June 2005
- The Evolution of the UHMWPE: New Trends in Orthopaedic Applications.
L. Costa
7° E.F.O.R.T. European Congress 2005 - Lisboa, Portugal - 4/7 June 2005
- From Small to Big Diameters: Advantages, Opportunities and Solutions.
F. Benazzo - G. Rinaldi - F. Ravasi - P. Dalla Pria
7° E.F.O.R.T. European Congress 2005 - Lisboa, Portugal - 4/7 June 2005

For further information about our products, please visit our web site www.lima.it

This documentation has been written by Lima-Lto Sales & Technical Department

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X Lima Deutschland GmbH