Delta

Acetabular Cups
Coppe acetabolari

**SURGICAL TECHNIQUE
TECNICA OPERATORIA**

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Nota: Coppe dia. 50 e 52 mm Large non abbinabili a inserti Biolox
Note: Cups dia. 50 and 52 mm Size Large not suitable for Biolox Liners

Delta-FINS Cup
Coppa Delta-FINS

Delta-ST-C Cup
Coppa Delta-ST-C

Delta-FINS Acetabular Cups Coppe acetabolari Delta-FINS

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Nota: Coppe dia. 50 e 52 mm Large non abbinabili a inserti Biolox
Note: Cups dia. 50 and 52 mm Size Large not suitable for Biolox Liners

Acetabular Cups Delta-ST-C Coppe acetabolari Delta-ST-C

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UHMWPE X-LIMA + Ti6Al4V

| | size taglia | for heads dia. per teste dia. |
|-------------|------------------------|--|
| 5885.51.055 | S | 28mm |
| 5885.51.058 | M | 28mm |
| 5885.51.158 | M | 32mm |
| 5885.51.060 | L | 28mm |
| 5885.51.160 | L | 32mm |
| 5885.51.260 | L | 36mm |

Protruded Liners for Heads Dia. 28 mm, Dia. 32 mm and Dia. 36 mm
Inserti protrusi per teste dia. 28 mm, dia. 32 mm e dia. 36 mm

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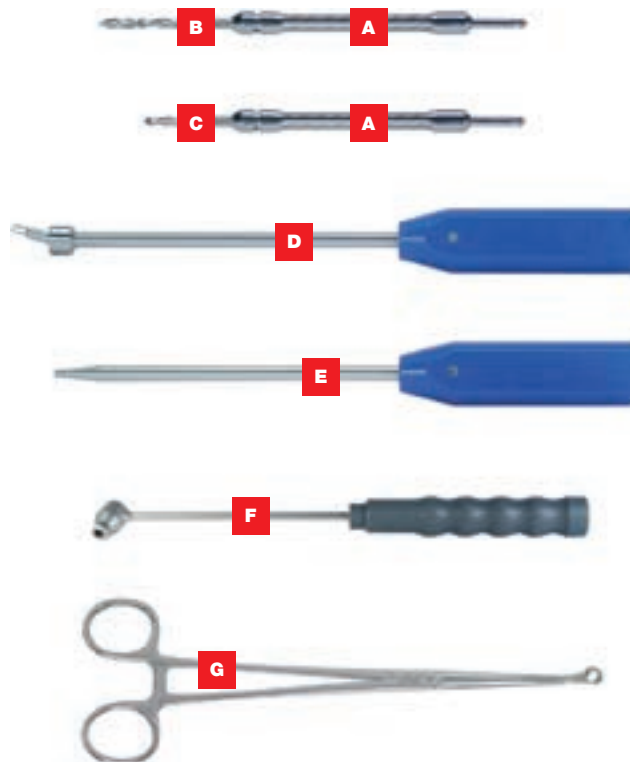
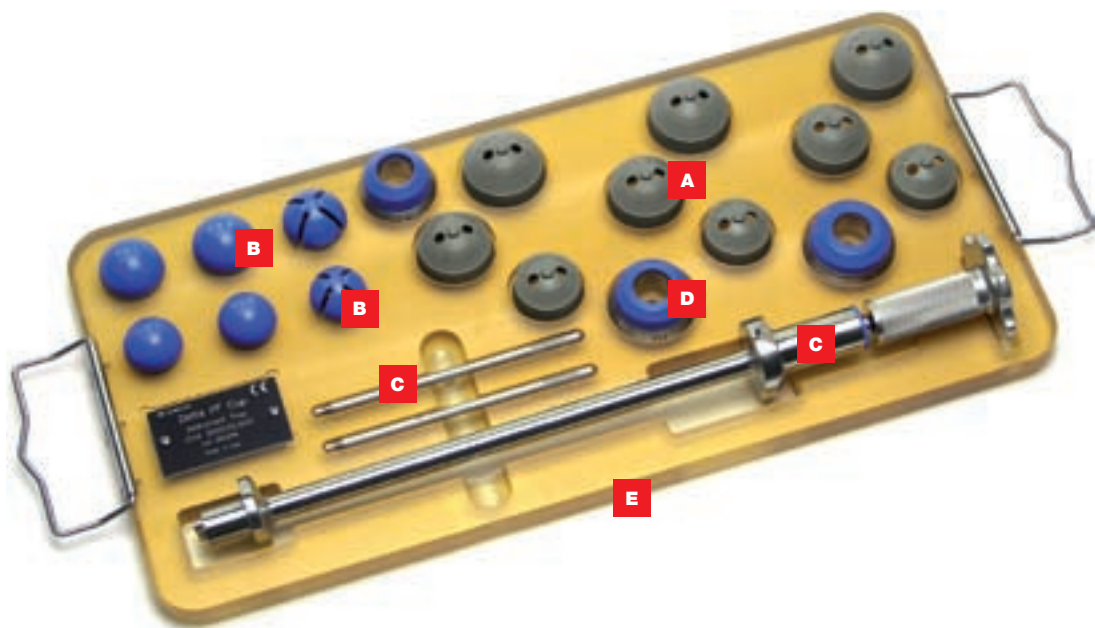
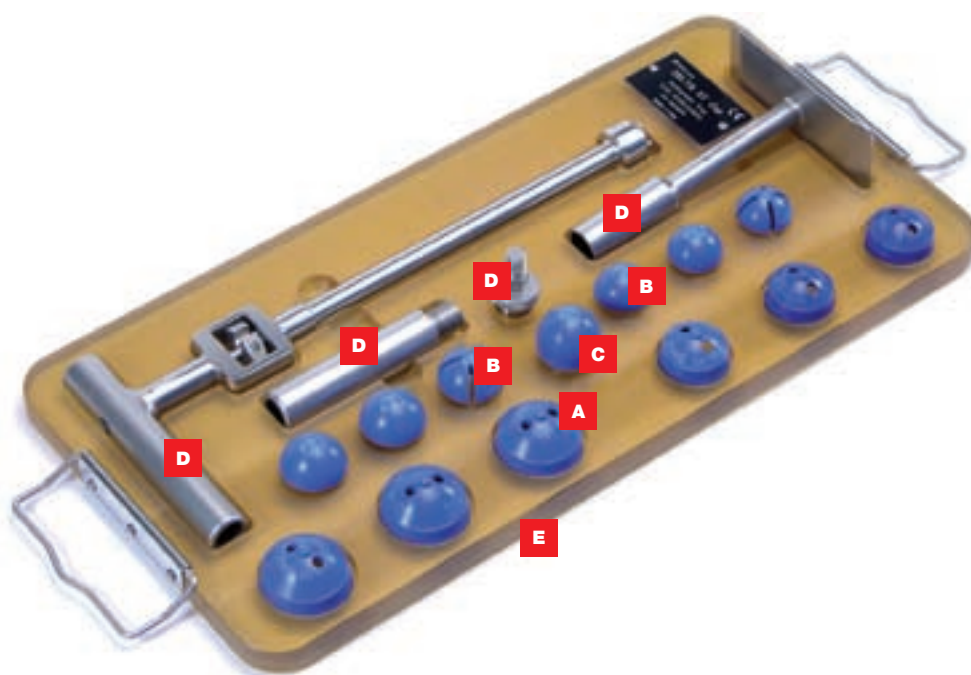
| ref. | description | descrizione | qt. |
|-------------|---|--|-----|
| 9055.50.020 | A Trial Liner, Large for Head Dia. 28mm | inserto prova Large per teste Dia. 28mm | 1 |
| 9055.50.025 | A Trial Liner, Large for Head Dia. 32mm | inserto prova Large per teste Dia. 32mm | 1 |
| 9055.50.030 | A Trial Liner, Large for Head Dia. 36mm | inserto prova Large per teste Dia. 36mm | 1 |
| 9055.50.220 | A Trial Liner, Small for Head Dia. 28mm | inserto prova Small per teste Dia. 28mm | 1 |
| 9055.50.225 | A Trial Liner, Small for Head Dia. 32mm | inserto prova Small per teste Dia. 32mm | 1 |
| 9055.50.230 | A Trial Liner, Small for Head Dia. 36mm | inserto prova Small per teste Dia. 36mm | 1 |
| 9095.10.521 | B Trial Head Low Taper 12/14 Dia. 32mm Short | testa prova cono ribas. 12/14 Dia. 32mm Short | 1 |
| 9095.10.522 | B Trial Head Low Taper 12/14 Dia. 32mm Medium | testa prova cono ribas. 12/14 Dia. 32mm Medium | 1 |
| 9095.10.523 | B Trial Head Low Taper 12/14 Dia. 32mm Long | testa prova cono ribas. 12/14 Dia. 32mm Long | 1 |
| 9095.10.531 | B Trial Head Low Taper 12/14 Dia. 36mm Short | testa prova cono ribas. 12/14 Dia. 36mm Short | 1 |
| 9095.10.532 | B Trial Head Low Taper 12/14 Dia. 36mm Medium | testa prova cono ribas. 12/14 Dia. 36mm Medium | 1 |
| 9095.10.533 | B Trial Head Low Taper 12/14 Dia. 36mm Long | testa prova cono ribas. 12/14 Dia. 36mm Long | 1 |
| 9057.20.320 | C Liner Impactor for Dia. 36mm Head | impattatore inserto per testa Dia. 36mm | 1 |
| 9055.21.010 | D Wrench for Cups DELTA-ST | chiave inserimento coppa DELTA-ST | 1 |
| 9055.21.900 | E Instrument Tray | vassoio strumentario | 1 |
| 9095.10.920 | Sterilizable Box | box sterilizzabile | 1 |

Delta-TT, Delta-PF and Delta-FINS Surgical Technique

Tecnica operatoria Delta-TT, Delta-PF e Delta-FINS

1. Milling of the Acetabulum

Using the desired surgical technique, expose the acetabulum so as to view it adequately for a correct reaming, remove possible osteophytes and expose the cotyloid rim thus obtaining an excellent view of the bone anatomy to verify the presence of cavitory and/or segmental defects.

The cotyle seat is prepared with the usual acetabular milling cutters.

Reamer with Cross Coupling
Fresa con attacco a croce
cod. 9057.20.942...

[1]

Reamer Handle EZ
Manipolo per fresa EZ
cod. 9057.20.811

[2]

[3]

[4]

Start reaming the acetabulum with a milling cutter (Reamer with cross coupling fig. 1), of a diameter smaller than that displayed on the templates, mounted on the milling cutter holder.

Insertion of the spherical milling cutter into the reamer handle (Fig. 2):

- 1) Insert the milling cutter next to the pegs fitted on the locking lever;
- 2) Push the cutter handle locking guide downward and turn the milling cutter anticlockwise by 1/4 turn (fig. 3).
- 3) Then release the locking guide (fig. 4).

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[5]

Insert the milling cutter into the acetabulum by keeping it on an abduction axis of approximately 45° (fig. 5) and an antversion of 15°; operate the attached manipulator clockwise.

To remove the milling cutter, push the locking guide again and rotate the milling cutter anticlockwise to remove it from the handle.

Proceed gradually with larger diameter milling cutters until reaching the subchondral bone.

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[6]

2.Trial and acetabular cup introduction

Once the acetabular seat preparation phase has been completed, use the trial cup (fig. 6) of the size corresponding to the last milling cutter employed. Hang it onto the positioner or onto the multipurpose handle.

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[7]

Each trial cup have the same diameter as that of the milling cutter, there being no interference since their function consists in evaluating the accuracy of the prepared seat.

Check the contact onto the bone wall throughout the wide windows (Fig.7).

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The choice of the cup model PF or FINS follows the pathology.

The use of the FINS model is suggested in case of CDH or early primary revision.

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[10]

[11]

At this point a cup having the same nominal diameter as that of the last milling cutter used is implanted. For example, if a 54mm diameter (actual diameter) milling cutter was used, a 54 mm nominal diameter cup must be implanted. This is the diameter shown on the package and marked on the prosthesis. The real cup diameter is of 56 mm so as to obtain an interference with the bone seat of to 2 mm.

To introduce and impact the cup a special wrench for Delta-PF cups (Fig. 8) is available. The wrench is equipped with three modular adaptors, sizes S,M,L like acetabular liners, which optimize the impact stress distribution at the cup (Fig.9)

Choose the adaptor in according to the cup size (size is marked onto the packaging label as well as into the inner bottom of the cup) and assemble to the terminal end of the wrench in correspondence with the two pegs and thread (Fig 10).

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
The adaptor is magnetically clamped if the way of assembling is right. Draw the cup close to the wrench. (Fig 11)

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[12]

Relatively rotating the cup and the wrench, look for the hanging position in the way the rim of the cup is completely in contact with the adaptor periphery.

During the manoeuvre, a slight snapping of settlement advises the two pegs lodge into polar site. The opposite handling end of the wrench advises as well a correct hanging position.(Fig. 12-13).

Tight firmly the wrench keeping steady the cup.(Fig.14)

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[15]

The special design of the wrench with the adaptor and the inner profile at the bottom of the cup make a strong hanging which avoids any risk of accidental disengaging during the introduction and the impaction.(Fig.15)

Moreover the adaptor facilitate to refasten the cup

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[16]

Place the cup in the acetabulum with an angle of abduction of about 45° (Fig. 16).

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Warning! If a ceramic liner might be used it is essential for the safety of the device that the covering angle of the cup does not exceed 45° (an angle of 40° is better) and that the angle of ante-version is between 10° and 20° (Fig. 17).

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[17]

A mistake in the positioning could cause damage of the ceramic liner if sub-dislocation of the femoral head occurs. This is also why it is extremely important to check the correct ante-version of the femoral stem.

If it is thought that bone screws will be used, position the cup in such a way that the three holes for the screws (closed by their appropriate screw caps) are in the superolateral area.

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The instrumentation set support specific tools to control the positioning of the acetabular cup with good precision, no matter what the position of patient and the surgical approach are used.

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[18]

Insert the collimation bars in their threaded holes (Fig. 18) on the shaft of the wrench.

For instance if the patient lies on lateral decubitus, it is possible to check that if the first bar (A) is horizontal and the second (B) is perpendicular to the surgical bed, the cup is in 45° abduction and 15° anteversion (Fig. 19).

Hold the wrench steady and hit along the axis with a hammer heavy enough to tap the cup firmly into the bony socket. Check that the cup is sunk into the acetabulum sufficiently and that the implant is initially stable by delicately levering the wrench shaft in various planes.

Disengage the wrench just by unscrew the knob and check the contact between the cup and the acetabulum through the polar and cranial holes (removes one or more plugs). If necessary refasten the wrench and repeat the axial hammer taps.

If the primary stability of the cup is judged to be insufficient, independently of the use of further bone screws, further milling of the acetabular site is recommended, using a reamer of larger diameter and thus its corresponding cup.

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[24]

[25]

If required, further bone screws can then be inserted (the holes of cup must be in the supero-lateral position). Using the **jointed** screwdriver as before or one (Fig. 20), remove one or more of the threaded caps (Fig. 21), that is, as many as the bone screws that are intended to be used and rescrew the pole cap.

3. Introduction of the Bone Screws

If you wish to use bone screws, we recommend that you use exclusively the bone screws supplied with the system (Fig. 22). Other screws could create problems with the correct insertion of the articular liner in the metal cup.

Introduce the **drill bit** (Fig. 23) with a **flexible mandrel** into the **drill guide** (Figs. 24-25).

Two **helix drills** are available, one **short** and one **long** (Fig. 23), which make holes of 15 and 30 mm, respectively.

Place the drill guide in the selected hole of the cup respecting the direction of the hole and then drill the bony tissue (Fig. 25).

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[30]

Grasp the bone screw with the **pliers for screws** (Fig. 26) below the spherical head and start screwing into the bone using the fixed or jointed screwdriver (Fig. 27).

As soon as the screw starts entering the bone, remove the pliers and complete the screwing (Fig. 28).

The head of the screw must not protrude obviously from the interior of the acetabulum; if it did it might prevent the articular liner from stopping in the right place. If necessary repeat this procedure for the other holes. No more than three screws can be used.

4. Trial Reduction

Normally the acetabular procedure precedes the femoral one, so that once the acetabular component has been inserted it is recommended that all contact between the acetabular cup and femoral stem is avoided. In order to facilitate this, it is suggested that you insert into the cup the trial liner, which can also be used in trial reduction of the hip replacement, or a protection tampon.

The use of a **trial liners** (Fig. 29) is advised to check the joint movement.

The instrument set for Delta-PF Cups provides also the **trial heads with low taper** (Fig. 30) with a diameter 32 and 36 mm to perform the trial reduction with Lima-Lto femoral stems (Figs. 31-32).

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[31]

If the cup is poorly positioned in relation to the femoral component (risk of impingement or dislocation) avoid using ceramic liners.

Choose a protruded polyethylene liner.

[32]

Reply onto the bone a mark by a bovie in correspondence to the preferential way of dislocation.

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Metal Liner
Inserito metallico

5. Insertion of the Definitive Articular Liner

Before inserting the definitive articular liner (whose size is printed on the packaging of the acetabular cup) clean the interior of the cup carefully and check that the surrounding soft tissues do not prevent the prosthesis from entering it.

NB: screw the polar plug, which is available together with ceramic or metal liners.

The polar plug is not necessary when a polyethylene liner is used, because a it is already equipped by a Titanium alloy polar shielding (Fig.33-38)

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[33]

All the liners of Delta System are fixed by a Morse taper. This does not require either snap devices or antirotation pegs on the external edge of the cup.

The liner can, therefore, be inserted in any position

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[34]

Use of the ceramic liner

The ceramic liner is inserted by the positioner (Fig. 34).

The positioner is equipped with three peripheral elastic grips, which hold up the liner during the insertion, and allow to lay it down onto the cup rim aligning the position relatively at the equatorial plane.(Fig.35)

Push on axis the little knob with a finger. The liner is disengaged and release into the cup inner seat.(Fig.36)

[35]

Check for the right liner lodging just touching all the rim perimeter. The rim of the liner must not be out the cup one, otherwise there might be liner breakage or chipping (Fig. 37).

[36]

In the case the liner were placed across and blocked in, it is necessary to be removed, stroking the metallic edge of the cup by an osteotome with flat blade. The vibrations pop the liner out of the socket.

[37]

Attention! Do not re-connect the removed ceramic liner, nor insert a new one into a cup lodged a ceramic liner.

Complete the implant only and exclusively using a liner in polyethylene (normal or cross-linked) and, when possible, adopt exclusively a ceramic femoral head, otherwise the femoral prosthetic device must to be as well revised with a new one. Then continue like indicated.

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UHMWPE Liners
Inserti in UHMWPE

(38)

Use of the polyethylene (Fig. 38) and metal liners

Nota. UHMWPE liners are just equipped by a polar peg which is coated of titanium. Therefore there is not need of polar plug.

For polyethylene and metal liners it is also possible to perform the manoeuvre of connection without the positioner, therefore handy inserted keeping it between the thumb and forefinger. The forefinger is straight into liner concavity

Release in the cup the liner gently pushing into by the forefinger. Check for the right liner lodging as well done for the ceramic liner insertion.

The removal of the metallic liner must be executed with the same modalities of the ceramic one.

The removal of the polyethylene liner is executed screwing at the inner bottom a self-tapping screw for bone.

Note. For the protruded liners check the position of the antiluxation rim before blocking the liner definitively in its site.

A slight mark (Fig.38) engraves that rim to allow the liner orientation with respect to the marked reference of trial.

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Liner Impactor
Impattatore inserto
cod. 9057.20.300... [39]

To assure stability coupling, screw the **liner impactor** (Fig. 39) onto **the beater** (Figs. 40-41) and gently tap the liner in an axial direction (Fig. 42).

The bearing load will definitely lock the coupling

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Multi Purpose Handle
Manico multiuso
cod. 9055.32.025

At the end clean and wipe the coupling surfaces carefully before definitively reduce the joint.

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[42]

[43]

6. Trial components and reduction for metal on metal implant

Assemble the plastic trial adaptor taper, of the desired size, on the femoral stem taper. (Fig. 43).

[44]

Then, assemble the plastic trial head, in a size matching the implanted cup, on the adaptor taper and carry out the trial reduction (Fig. 44).

[45]

7. Definitive implant (MoM)

Install the final adaptor taper, for the corresponding head diameter, on the 12/14 taper of the implanted femoral stem, and fasten it tightly.

On the adaptor taper, assemble the final head of the chosen diameter and fasten it tightly.

It is also possible to assemble the head/adaptor taper implant outside the surgical field by impacting the Morse taper with a beater. Protect the components with a gauze and assemble them with a hammer (Figs. 45-46).

[46]

8. Trial component for double mobility

It is also possible to fit a large diameter polyethylene mobile liner in the same cup so that it articulates with an internal head of 28 mm.

Use the head to test the diameter that can be matched to the cup previously

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[47]

implanted and perform the trial reduction (Fig. 47).

Note. You should not use the 28 mm skirted femoral heads with the double mobility PE liner.

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9. Final implant with double mobility

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[48]

Take the 28 mm head (in ceramic or metal) and the polyethylene head that matches the implanted cup.

Position the components in the press (Fig. 48) and turn the screw until the two components have been completely assembled.

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Insert the components on the taper of the stem and perform the final reduction of the implant (Fig. 49).

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Delta-ST-C Surgical Technique

Tecnica operatoria Delta-ST-C

Follow the same indications for **Mill-
ing of the Acetabulum** described at
paragraphes 1 and 2 of the Delta PF/
Delta-FINS technique.

Being a self-tapping design, ST and
ST-C are implanted by screwing. The
reaming is completed with a slight
peripheral flaring of the acetabular site
to allow the threading.

At this point the cup having the same
real diameter as that of the last milling
cutter used is implanted.
For example, if a 54mm diameter
(actual diameter) milling cutter was
used, a 54 mm diameter cup must
be implanted. This is the diameter
shown on the package and marked
on the prosthesis. For this cups
there is not any interference with the
bone seat.

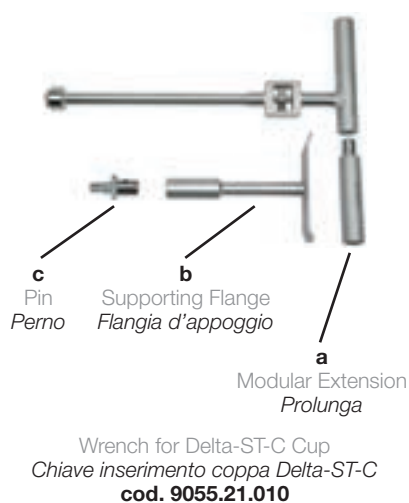
Bone screws will be not necessary for
the cup stability.

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[4]

1. Acetabular cup introduction

Insert the **wrench for Delta-ST-C cups** (Fig. 1) into the polar site, keeping the four pegs in the proper location (Fig. 2).

Clockwise screw the locknut of the handle (Fig. 3), locking the cup to the wrench (Fig.4).

Up to this point, the surgical steps are the same both for the postero-lateral approach with patient lying on lateral decubitus and direct approach with patient in supine position.

This page refers to the postero-lateral approach.

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[5]

a) postero-lateral approach with patient lying on lateral decubitis

Screw the modular extension (a) (Fig. 1) into the threaded hole which is on short portion of the wrench handle (Fig.5).

Insert the cup into the acetabular site and clockwise screw pushing on axial direction (Fig. 6).

When necessary, stabilize cup position at introduction by gentle hammer blows in the direction of wrench axis.

[6]

Note. For a right cup positioning read with care the next chapter before cup screwing.

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[7]

b) direct approach with patient in supine position

Wrench assembling:

1) Screw the modular extension (a) (Fig.1) into the threaded hole which is on the long portion of the wrench handle (Fig.7)

2) Screw the pin (b) (Fig.1) into the threaded hole which is on the handle of the wrench (Fig.8)

3) Connect the supporting flange (c) (Fig. 1) to the pin (Fig. 9). The flange turns freely around when coupled with the pin.

[8]

For the cup introduction, hold the wrench as in (Fig. 10). The flange leans on the Operator's bust.

The Operator grasps the shaft and the handle of the wrench and gentle pushing by the bust in axial direction.

The flange allows the operator to control cup anteversion keeping steady the desired one during the cup screwing.

Screw for half of a round the cup driving bust pushing in axial.

[9]

[10]

Note. For a right cup positioning read the next the next chapter before cup screwing.

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[11]

2. Positioning of the Acetabular Cup

Place the cup in the acetabulum with an angle of abduction of about 45° (Fig. 11).

Warning! If a ceramic liner might be used it is essential for the implant safety of the device that the covering angle of the cup does not exceed 45° (an angle of 40° is better) and that the angle of anteversion is between 10° and 20° (Fig. 12).

A mistake in the positioning could cause damage of the ceramic liner if subdislocation of the femoral head occurs. This is also why it is extremely important to check the correct anteversion of the femoral stem.

The general instrumentation set nevertheless allows to control the positioning of the acetabular cup with good precision.

After have temporarily remove the wrench, hang the positioning handle and introduce the collimation bars in their threaded holes(Fig.13).

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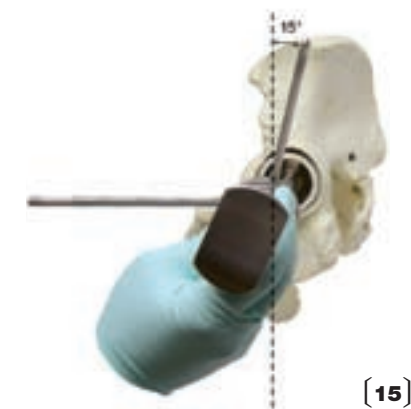
[13]

[14]

For instance if the patient lies on lateral decubitus, it is possible to check that if the first bar (A) is horizontal and the second (B) is perpendicular to the surgical bed, the cup is in 45° abduction and 15° anteversion (Fig. 14-15).

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[15]

Check the cup is enough subsided into the acetabulum and that the implant is initially stable by delicately levering the wrench shaft in various planes.

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[16]

Counter-clockwise screw the locknut on the handle for wrench releasing (Fig. 16) check the contact between the cup and the acetabulum through the inspection holes of the ST-C cup. If necessary rescrew the shaft and repeat the screwing procedure. If the initial stability of the cup is judged to be insufficient, further milling of the acetabular site is recommended, using a reamer of larger diameter and thus its corresponding cup.

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[17]

For an easy locknut unscrewing a collimation bar can be used (Fig. 17). Just put the threaded end portion of a bar into a hole on the locknut and, counter-clockwise pushing, turn it (Figs. 18).

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[18]

3. Trial Reduction and Insertion of the Definitive Articular Liner

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[19]

NB: screw the polar plug, which is available together with ceramic and metal liners (Fig.19-20). The polar plug is not necessary when a polyethylene liner is used, because it is already equipped by a Titanium alloy polar shielding.

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The liner can, therefore, be inserted in any position.

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Follow paragraphs 4 and 5 of Delta-PF technique how to perform trial and definitive liner insertion

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[20]

For further information about our products, please visit our web site www.lima.it

This documentation has been written by Lima-Lto Sales & Technical Department

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X Lima Deutschland GmbH
X Lima Austria GmbH